T150 - Process volume capacity 150 L per cycle.





The contaminated waste is loaded into the upper chamber where a heavy-duty shredder reduces the waste into small pieces. Using gravity, the processed material drops into the lower treatment chamber.



After shredding, waste and all the inner parts of the machine are steam heated to 138°C (280 F) and pressurized to 3,5 bar (51 psi).



Sterilization is achieved by maintaining a temperature of 138° C (280 F) at the core of the waste for 10 minutes.



After the cooling process, the final product is safe to recycle or to dispose as ordinary waste. Waste are both sterilized (Microbial inactivation = 10°) and volume-reduced by up to 80%.



The 30 minutes duration of an average cycle process is fully automated and totally monitored. A computerized control system scans the process and automatically prints and records on a memory card a batch report at the end of each cycle with the essential sterilizing parameters for accurate record keeping.



Approved by the Higher Council of Public Health of France under the direction of the Ministry of Health and the Ministry of the Environment.

ASME









T150 - Process volume capacity 150 L per cycle.





Technical specifications

General Characteristics

Dimensions (L x W x H)	274 x 200 x 300 cm 6,9 x 7,2 x 9 ft
Shipping Weight	800 kg
Max weight when filled with water For a special hydraulic test	1100 kg
Stress	1 kg/cm ²
Steam Pressure	8 bar
Max Steam Flow	65 kg/h
Compressed Air	6 bar
Electricity 380 V / 3-Phase	8 kW


Operating Characteristics


Average Cycle Time	30 minutes
Process Volume Capacity	150 liters
Average Waste Density	100 - 150 kg/m ³
Average Process Weight Capacity	15 - 23 kg/cycle 33 - 51 lb/cycle
Microbial Inactivation	10 8
Waste Volume Reduction	80%

Consumption / cycle

Steam	9 kg
Electricity	1,4 kWh
Water	15 liters



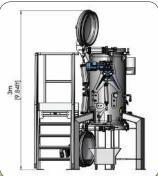

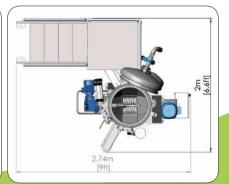

28 rue Sébastopol 59100 ROUBAIX - FRANCE Tél. +333 20 70 98 65 Fax +333 20 36 28 05 contact@ecodas.com www.ecodas.com

